



How to clean your Rigid Endoscope

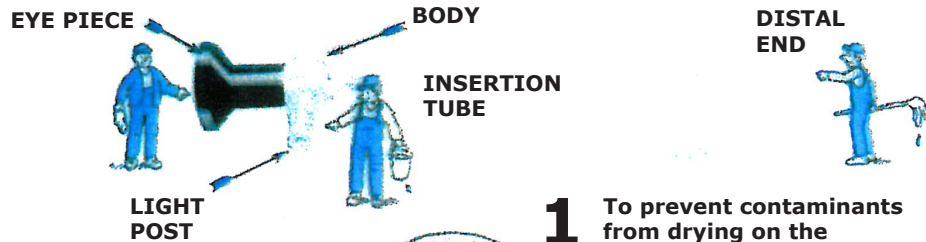

1 To prevent contaminants from drying on the endoscope, they should be cleaned, if possible, immediately after use.

2 All surfaces should be cleaned with warm water and mild soap. Never scratch dirt off with hard objects.

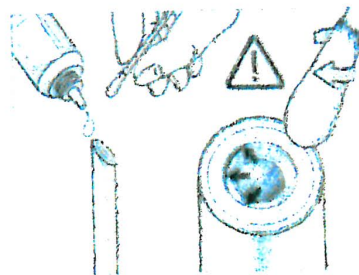

3 If stains are still present, a mixture (30:70) of methyl alcohol and acetone may be used. A fine woven cloth or lens tissue should be used for cleaning. Dry with a soft woven cloth.



4

Due to insufficient cleaning of foreign matter prior to sterilization, irremovable deposits may form on the three optical surfaces.

5

These deposits can easily be removed using the bio-compatible polishing paste which can be obtained from HBL.



6

To remove the deposits, dab some polishing paste onto a cleaning cotton tipped swab. Press gently onto the optical surface and clean in circles.



7

Rinse the optical surfaces with tap water to remove any remaining polishing paste and clean the surfaces as instructed.

